

Neurotransmitter in Urine Basic (F500) + Plus (F510) + Serotonin (F550) + Adrenaline (F600) + D-Arabinitol (A125) + Organic Acids (E408) + Marker of Vitamin metabolism (E435)



Test Instructions

The test set contains the following materials:

1 x request form

1 x instructions

1 x pipette for transferring the urine into the urine tube (white lid)

1 x urine tube with a white lid (contains a stabilizer = white substance)

1 x outer container (brown)

1 x sticker for labelling the urine tube

1 x shipping envelope

An urine collection cup can be ordered at biovis' by fax with the order form shipping material Art.-Nr. 63

Instructions:

The urine tube has to be filled with the **second morning urine**.

- Collect the second morning urine in the urine collection cup (or in a suitable clean vessel) after getting up.
- Tap lightly the bottom of the urine tube (white lid) several times onto a solid surface, so that the (non-poisonous) stabilizing powder accumulates at the bottom.
- Fill the urine tube (white lid) with the enclosed pipette.
(Please make sure that the stabilizing substance remains in the tube)
- Now place the urine tube in the brown outer container.
- Then the urine collection cup and pipette and can be discarded.

Please be sure to label the urine tube with:

- **name**
- **date**
- **material = tick „urine“ on the request form**

Please place the labelled urine tube in the brown outer container (protective container for the postal dispatch) together with the completed request form in the shipping envelope.