

## Tryptophan and histamine in stool (A410, A651, A670)



### Test Instructions

The following material is contained in the test set:

- 1 x request form
- 1 x instruction
- 1 x stool sample preparation system
- 1 x transport tube
- 1 x shipping bag

### Application of the stool sample tube

#### 1. Remove dipstick

Open the **upper yellow part** of the cap and remove the attached dipstick. The blue cone-shaped insert remains on the tube!

#### 2. Obtain stool sample

Insert the dipstick into the stool at 3 different sites. Make sure that the notches at the lower part of the stick are covered completely with stool.

#### 3. Place dipstick back into the tube

Stick the dipstick with the attached stool **only once** back into the sample tube. Excess material will be stripped off by the cone-shaped insert, letting 15 mg of sample into the tube.

#### 4. Prepare stool sample suspension

Close the tube properly by turning until you hear the yellow shutter click. Then shake well until no stool sample remains in the notches and the stool is completely suspended in the extraction buffer. The suspension now contains 15 mg of stool.

**Attention: The stool dilution in the sample tube hardly changes colour.**  
**Repeated insertion of sample into the tube will falsify the results!**

