

KMO Activity in Urine (A690) TMA and TMAO Formation in Urine (A675) Tryptophan Metabolism Plus (A685)



Test Instructions

Test Set includes:

- 1 x Request Form*
- 1 x Test Instructions*
- 1 x Centrifuge Tube stabilized*
- 1 x Transfer Pipette*
- 1 x Patient Label*
- 1 x Outer Transport Tube*
- 1 x Shipping Envelope / Return Envelope*

Instructions:

The urine tube with white cap must be filled with the **second morning urine** up to the mark above.

- When you wake up, catch the second morning urine in the morning urine cup (or in a suitably clean container).
- Use the enclosed pipette to fill the urine tube with the white cap up to the mark at the top - approx. 5 mm below the closed lid. Please ensure that the stabilizer substance remains in the tube
- The plastic cup and pipette can then be disposed of.

Please be sure to label sample containers with:

- **Name**
- **Date**
- **Material = tick the urine**

Please put the labeled urine tube into the white transport tube (protective container for mailing) and put it into the shipping envelope together with the completed request form.