

## Kryptopyrrol (L 120)



### Test Instructions

#### Test Set includes:

- 1 x Request Form**
- 1 x Test Instructions**
- 1 x Outer Tube light protected**
- 1 x Urine Monovette**
- 1 x Shipping Envelope / Return Envelope**

#### Instructions:

The Sarstedt-Urine-Monovette must be filled with **morning urine**. For an accurate result it is important to use the entire morning urine, i.e. the bladder should not have been emptied in the last six hours before.

- After getting up, catch the **first morning urine** in the urine collection cup. (or in an appropriately clean container).
- Fill the Sarstedt-Urine-Monovette (to do this, unscrew the small cap, put on the yellow plastic tip, pull on the piston until the tube is completely filled and then break off the piston at the predetermined breaking point, remove the plastic tip and unscrew the cap again).
- Insert the urine tube into the brown outer packaging tube.
- Plastic cup and pipette can then be disposed.
- Please label the enclosed label and stick it onto Sarstedt-Urine-Monovette.

#### Please be sure to label sample containers with:

- **Name**
- **Date**
- **Material = tick urine**

Please put the labelled urine tube into the black transport tube (protective containers for postal dispatch) and put these into the shipping envelope together with the completed request form.

**Important:** Send the test set by post as quickly as possible. Shipment over the weekend and public holidays should be avoided.