

Neurotransmitter in Urine Basic (F500) + Plus (F510) + Serotonin (F550) + Adrenaline (F600) + D-Arabinitol (A125) + Organic Acids (E408) + Marker of Vitamin metabolism (E435)



Test Instructions

The test set contains the following materials:

1 x request form

1 x instructions

1 x pipette for transferring the urine into the urine tube (white lid)

1 x urine tube with a white lid (contains a stabilizer = white substance)

1 x transport tube

1 x sticker for labelling the urine tube

1 x shipping envelope

An urine collection cup can be ordered at biovis' by fax with the order form shipping material Art.-Nr. 63

Instructions:

The urine tube with the white cap must be filled with the **second morning** urine up to the mark at the top

- After getting up, collect the second morning urine in the urine collection cup (or in an appropriately clean container).
- Use the pipette provided to fill the urine tube with the white cap up to the mark at the top - approx. 5 mm below the closed lid. Please make sure that the stabilizer substance remains in the tube.
- The plastic cup and pipette can then be disposed

Please be sure to label the urine tube with:

- **name**
- **date**
- **material = tick „urine“ on the request form**

Please place the labelled urine tube in the transport tube (protective container for the postal dispatch) together with the completed request form in the shipping envelope.