

ColoAlert - Test for the early detection of colon cancer in stool (H205)




Test Instructions

Test Set includes:

- 1 x Request Form
- 1 x Test Instructions
- 1 x Stool Catcher
- 1 x Stool Tube for genetic analysis
- 1 x Stool Tube for occult blood test
- 1 x Transport Tube
- 1 x Shipping Envelope / Return Envelope

To ensure a high diagnostic certainty of the stool examination, please observe the following points when taking samples:

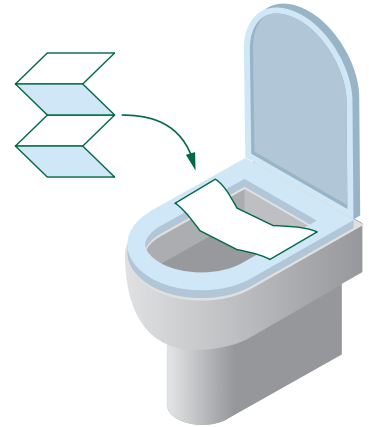


1. Stool sample taking for genetic analysis

- For stool catching please use the **enclosed stool catcher**. Stool may not come into contact with toilet bowl or water.
- Please unscrew the **stool tube** with **containing liquid** – **Attention: please do not pour out liquid!** Please use spoon to collect **in total a maximum of one pea sized amount of stool** at three different places - in the middle of stool sampling. **ATTENTION: The sample amount should NOT exceed the size of the spoon!**
- Screw the cover with the spoon back onto the **stool tube**. Shake the stool tube for at least one minute until an **even mixture** is obtained.
- Please write your **name, date of birth** and **date of sample taking** on the **label** and insert it into the enclosed transport tube.



2. Stool sample catching for occult blood test

- For stool catching please use the **enclosed stool catcher**. Stool may not come into contact with the toilet bowl or water.
- Please unscrew the **stool tube (brown lid)** and use the spoon to take samples from several spots. Please fill the tube **up to the marking**.
- Screw the lid with the spoon back onto the **stool tube**. Please write your name, date of birth and **date of sample taking** on the **label** and insert it into the enclosed **transport tube**.

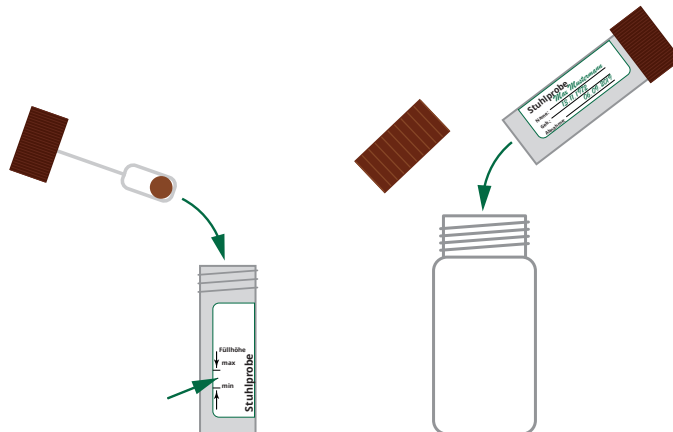

3. Shipping

- Please put the transport tube and the filled request form in the envelope supplied and preferably bring it to a post office. **Important: Please do not post on a Friday (weekend) or before a national holiday!!!**