

Estrogen Metabolites (F325)



Test Instructions

Test Set includes:

- 1 x Request Form**
- 1 x Test Instructions**
- 1 x Centrifuge Tube stabilized**
- 1 x Transfer Pipette**
- 1 x Patient Label**
- 1 x Transport Tube**
- 1 x Shipping Envelope / Return Envelope**

Instructions:

The urine tube with white cap must be filled up to the mark above with the second morning urine.

- After getting up, catch the second morning urine in the urine collection cup. (or in an appropriately clean container).
- Use the enclosed pipette to fill the urine tube with the white cap up to the mark above - approx. 5 mm below the closed lid. Please ensure that the stabilizer substance remains in the tube.
- Plastic cup and Pipette can then be disposed.

Please be sure to label sample containers with:

- **Name**
- **Date**
- **Material = tick urine**

Please put the labelled urine tube into the white transport tube (protective containers for postal dispatch) and put these into the shipping envelope together with the completed request form.